

INSTRUCTIONS FOR USE




NewGen Surgical O.R. Basins

Please read all information carefully.

Intended Use

The NewGen Surgical O.R. Basins and Bowls are sterile O.R. accessories designed to hold fluids and instruments during a surgical procedure.

www.newgensurgical.com/ifu

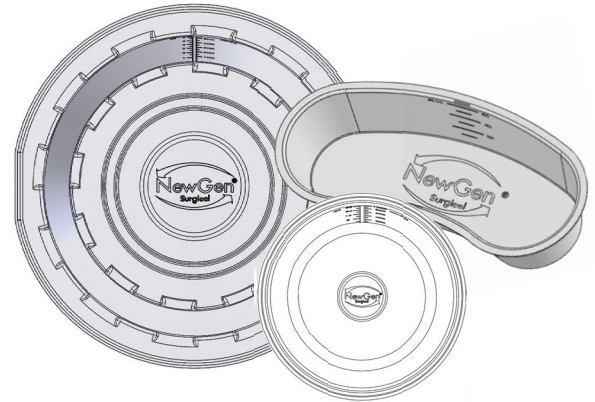

Warnings and Precautions

The NewGen Surgical O.R. Basins are manufactured with natural materials. With these materials, some discoloration and texture variation may be present; this does not affect the performance or sterility of the product.

To ensure performance, do not submerge in water or other liquids. Do not use excessive force.

This device is manufactured and sterilized for single use only.

Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure.

Store in a Cool, Dry Place.

Do not use if product is damaged. Do not use if package is open or damaged.

Instructions for Use

Follow recommended healthcare procedures.

	Sterilized using ethylene oxide		Catalog number
	See Instructions for Use		Do not use if package is opened or damaged
	Do not reuse		Latex free
	Batch code		DEHP free
	Use by YYYY-MM-DD or YYYY-MM		PVC free
	Manufacturer		

REF NGS5500RB, NGS700EB, NGS32SB



NewGen Surgical, Inc.
San Rafael, CA 94901 USA
855.295.4500
www.newgensurgical.com

© 2019 NewGen Surgical, Inc.
Manufactured in China

Patent Pending
45-3521-B